

SafeWinch


2 in one system
to ease your life

A new revolutionary system that will improve the life of wheelchair users and their carers.

It's a combined wheelchair winch and front tie down system made of two compact units which are synchronised and self-locking.

You will be able to easily transfer and secure a wheelchair into a vehicle with a simple use of a remote control.

CREATED & PATENTED by

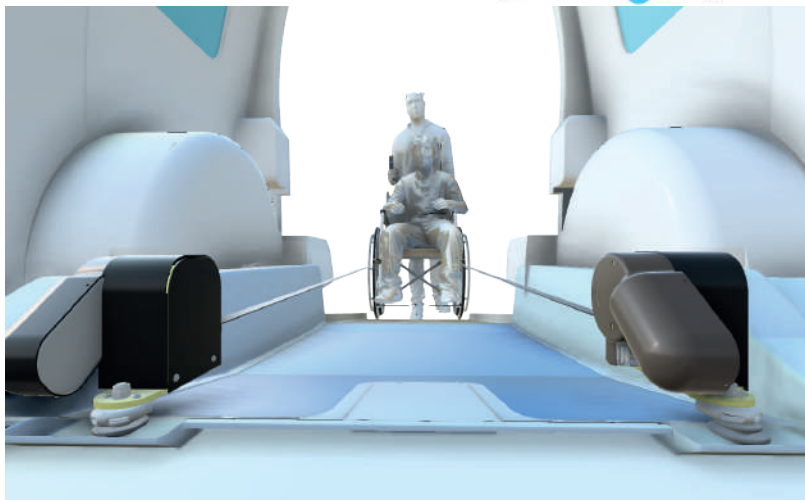

EASY TO USE - COMPACT - EFFORTLESS - SELF LOCKING - SYNCHRONISED

Two independent winches automatically synchronized by a central control unit.

The attendant only needs to attach the straps to the wheelchair and press: IN on the remote control to raise the wheelchair and OUT to lower it.

SafeWinch is used as front wheelchair tie-down, therefore it has been tested in accordance with ISO 10542.



10 Bis rue du 11 novembre - 80270 Airaines - FRANCE - contact@access-ability.eu - +33 (0)3 22 25 58 65

www.access-ability.eu



SafeWinch

2 in one system
to ease your life

Kit Contents

- 2 motor units (left and right)
- 1 central control unit
- 1 control panel
- Electrical harnesses
- 1 large remote control
- 1 small remote control



Going up into the vehicle

1. Position the wheelchair at the bottom of the ramp
2. Press OUT to unwind the straps and attach them to the wheelchair
3. Press IN to start the ascent while guiding the movement
4. Stop when the wheelchair is near its final position
5. Attach the rear tie-down manually
6. Press IN again to automatically tighten the front straps
7. Check that the LED blinks twice
8. Buckle the occupant's seatbelt
9. Close the rear door: the SafeWinch locks automatically

Going down from the vehicle

10. Press OUT for a few seconds to slightly release strap tension.
11. Unhook the rear tie-downs.
12. Keep pressing OUT to continue lowering the wheelchair while guiding it.

Pairing a New Remote

1. Open the hatch on the central control unit.
2. Press the programming button located under the hatch.
3. Press a button on the new remote to be paired.
4. The remote is now stored and ready to use.



10 Bis rue du 11 novembre - 80270 Airaines - FRANCE - contact@access-ability.eu - +33 (0)3 22 25 58 65

www.access-ability.eu

