

VEHICLE HARNESS



The new Access-Ability support harness ensures the safe transportation of passengers with behavioral disorders in a car. It is used to:

- Keep individuals with disabilities or behavioral disorders—such as autistic children or adults—in a seated position.
- Provide postural support for individuals with physical disabilities while seated in the vehicle's original seat.



With daily use, the lifespan is estimated at 10 years

Features

- Ergonomic design with height-adjustable backrest to suit body size.
- Made in France from tough, comfortable neoprene fabric
- Scalable and durable: several levels of strap adjustment
- Universal: easy to install, it adapts to any type of seat or booster seat

- Reimbursement by the MDPH possible under 'Technical aids'.
- Free trial in France and Belgium

- Lightweight: easy to take on trips.
- Secure: 7 lockable buckles.

IN AN EMERGENCY!

-Escape Box rapid evacuation box optional and recommended for emergencies



-Standard rapid emergency evacuation system



PRODUCT OPTIONS

- Emergency escape box -- Escape Box



- Set of 2 additional straps for individual seats



- Height-adjustable headrest for forward lateral support

INDICATIONS AND CONTRAINDICATIONS

- If the user is shorter than 1.10 meters, it is not recommended to use the Access-Ability support harness without a booster seat. Otherwise, they may experience discomfort around the neck area, and the device will be less effective.
- Therefore, it is not advised to use the Access-Ability support harness in any way other than as specified in the user manual.
- It is also not recommended to machine wash the support harness, as this may cause deformation and accelerate wear. If cleaning is necessary, use a damp sponge and/or a dry cloth.
- It is also not recommended to use the harness in direct contact with the skin (e.g., bare chest).
- The use of a seat belt remains mandatory when using the harness. The harness does not replace the seat belt.
- It is essential to regularly check the condition and compliance of this support harness when used in a vehicle, to prevent any incident. Any negligence in this regard may lead to risks, and the responsibility for ensuring proper function lies with the user.
- The seatbelt cutter hammer supplied with the harness must remain in the vehicle and be easily accessible in case of emergency.
- It is recommended that the user be accompanied throughout the journey.

INSTALLATION INSTRUCTIONS

Seat support base



Adjustable backrest



Breastplate or front waistcoat



BENCH SEAT INSTALLATION

- 1. Detach the chest plate from the support base and the backrest (A, B, C).
- 2. Slide the adjustable backrest (B) onto the support base (A). Place the combined unit (A+B) against the car seat backrest. If necessary, fold the car seat forward to pass the straps through and fasten them.
- 3. Tighten the straps around the seat back as much as possible.
- 4. The top of the support base can go either below or above the headrest, depending on the person's height.
- 5. Once the person is seated, clip the chest plate (C) onto the assembled base (A+B) and adjust the length of the 7 straps.
- 6. Fold back the strap ends to prevent them from slipping.



INSTALLATION ON INDIVIDUAL SEAT

The main stages of installation remain the same as for a standard seat.



An additional strap can be hooked on the back to extend the length of the seat.




A second strap is attached to the bottom of the seat back.



EASY EMERGENCY EVACUATION IN 1, 2 AND 3

1. person strapped in with the harness
- 2 The harness is released by pressing the 2 buttons on the back of the seat
- 3 The person can leave with the harness in case of emergency

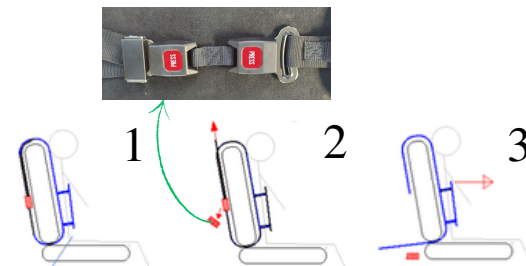

ACCESS ABILITY



Address : 10 bis rue du 11 novembre

80270 Airaines, France

tél: +33 (0)3 22 25 58 65

contact@access-ability.eu; www.access-ability.eu



UDI Unique Device Identifier

Use-By Date

MD Medical Device

CE CE Marking

LOT Manufacturer's Lot Number

REF Catalogue Reference

Manufacturer