





The smallest yet mightiest winch on the market

« **MicroWinch** » is part of our new wheelchair winch range available now in the market. The **MicroWinch** is an effective and functional device, developed to assist with the loading (and un-loading) of a wheelchair passenger into an adapted vehicle. It provides the necessary pulling power, making the process substantially less strenuous for the carer whilst maintaining passenger comfort at all times.

## VERTICAL VERSION - V

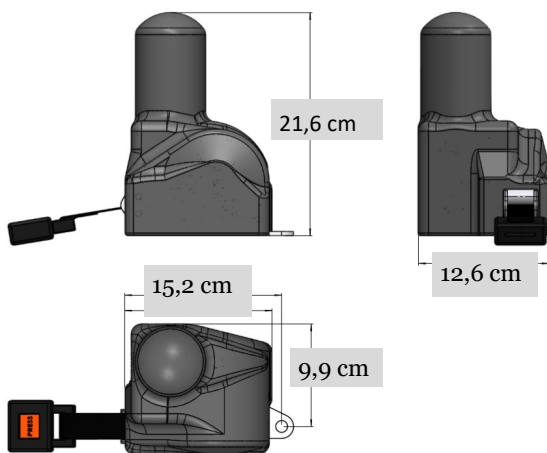

## HORIZONTAL VERSION - H

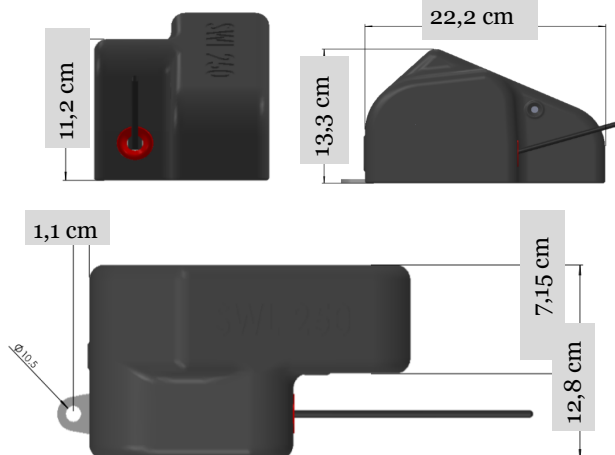

Powered by a high-performance electric motor and integral anti-roll back mechanism. It boasts an impressive safe working load capacity of 250kg and is enhanced with a range of safety features and benefits. Meticulously engineered and cleverly designed with a small-scale footprint, the MicroWinch is capable of fitting in any vehicle with limited interior space.

<b>Maximum load capacity:</b>	<b>250 kg</b>
<b>Weight:</b>	<b>3,4 kg</b>
<b>Belt lenght :</b>	<b>6 meters</b>
<b>Option:</b>	<b>3m belt extension</b>
<b>Warranty:</b>	<b>1 year</b>



### Included in the winch kit:

- 1 winch
- 1 small remote control
- 1 big remote control
- 1 V belt
- Fitting accessories



Address : BP19, 80490 Hallencourt, France


Tel : +33 (0)3 22 25 58 65

Mobile : + 33 (0) 6 45 72 46 90

[contact@access-ability.eu](mailto:contact@access-ability.eu)

[www.access-ability.eu](http://www.access-ability.eu)

## Wiring diagram:

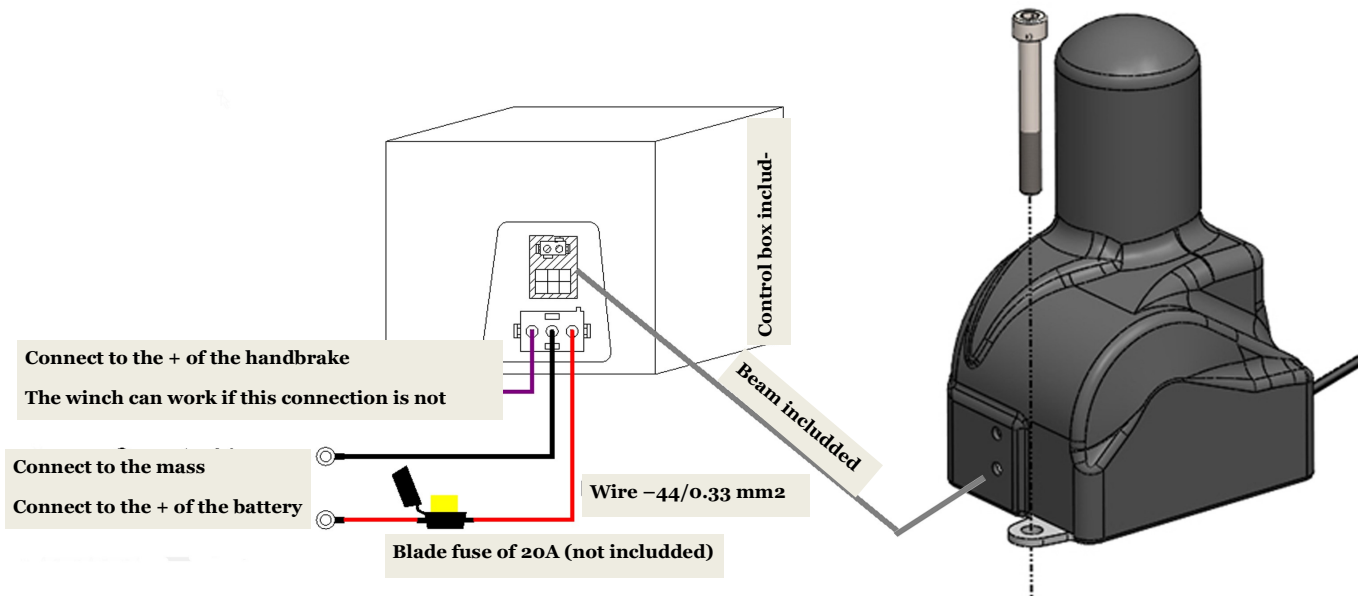

## Instinctive use:

Two **remote controls** are supplied with the winch. A big one, ideal for intensive use and an extra small one. Both work the same way.

1. Install and secure the winch in the vehicle.
2. Unwind the corde simply by holding down the "OUT" button. During the unloading, make sure that the wheelchair is stable when moving backwards.
3. Rewind the corde by holding down the "IN" button. For the loading make sure that the wheelchair is correctly positioned on the ramp. The winch will stop if the button is released. When the corde has reached its final end position, the winch will automatically stop.



It has a « **Gentle Start & Stop** » unique feature and a **remote control** providing **comfort and security to the** wheelchair user and assistant.

**Please read the product user manual before use!**

**Please contact us for a quotation**



Address : BP19, 80490 Hallencourt, France

Tel : +33 (0)3 22 25 58 65

Mobile : + 33 (0) 6 45 72 46 90

[contact@access-ability.eu](mailto:contact@access-ability.eu)

[www.access-ability.eu](http://www.access-ability.eu)